
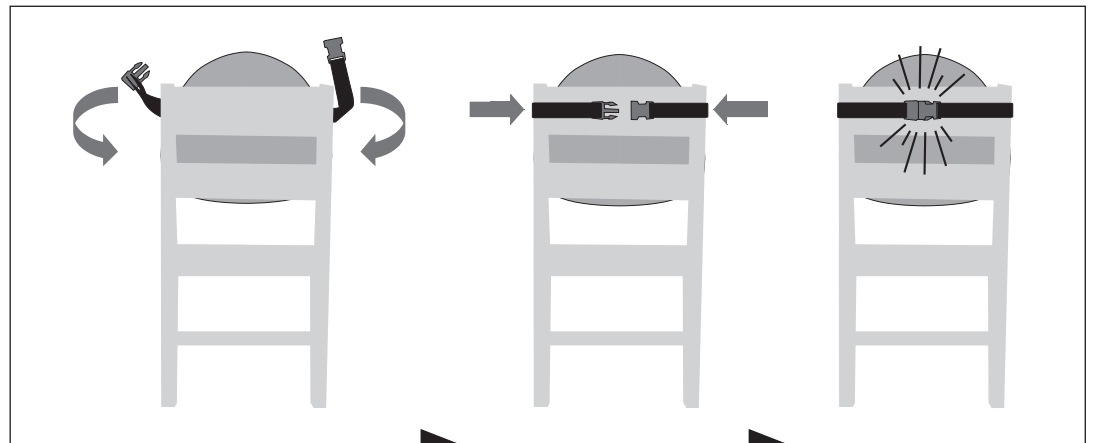




User Manual & Specifications

Liners: Size 1, 2, 3, 4, 5

Model # Series 72XXXXXX
UMLIN06-22LIT

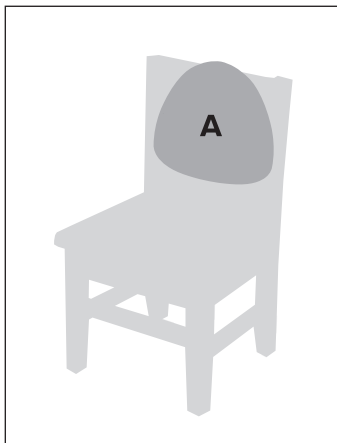

Secure product straps tightly around back of chair.

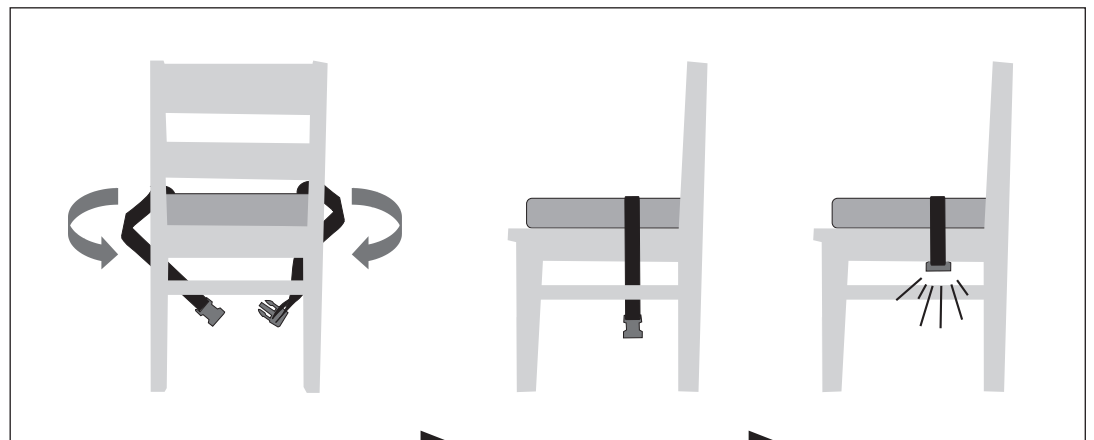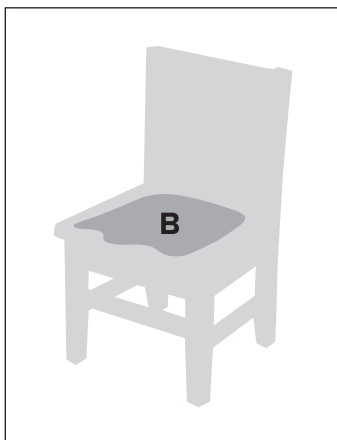


Secure product straps tightly around seat of chair.



Intended Use

The Special Tomato Soft-Touch® Liners offer an affordable alternative for children with mild to moderate seating support, postural, and therapeutic needs. They are made of Soft-Touch material that is latex-free and a 3-point pelvic harness is included. The Soft-Touch Liners can be attached to most seating systems including but not limited to the following: dining room chairs, restaurant chairs, classroom chairs, office chairs, strollers and wheelchairs.

Soft-Touch Liner Specifications

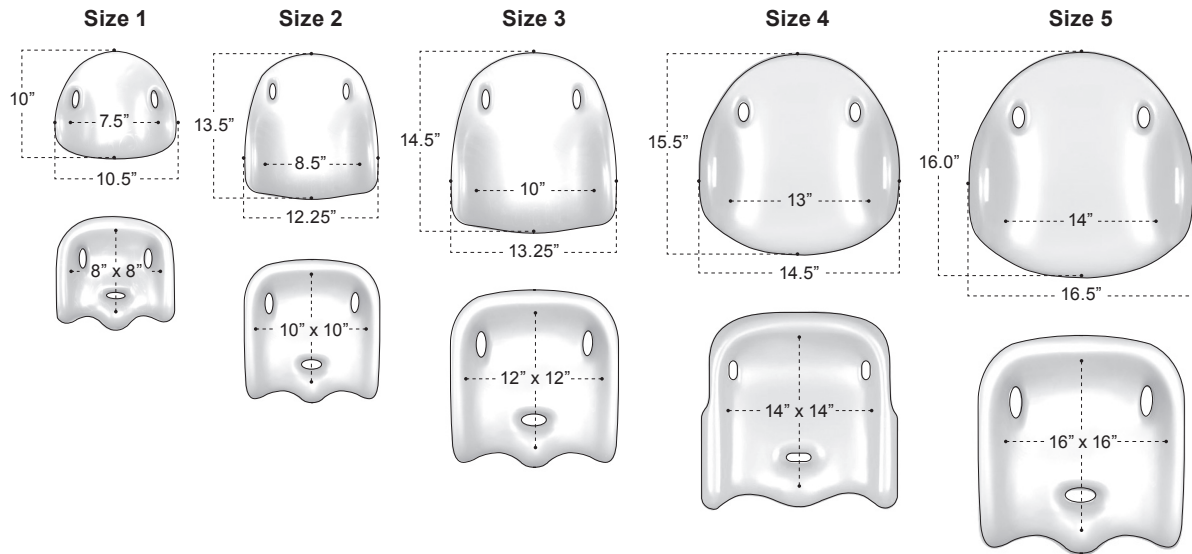

Hip-Flex Liner Specifications

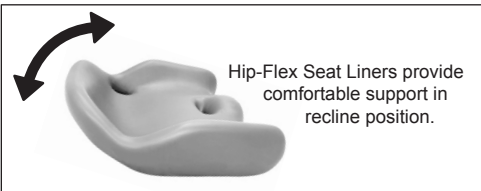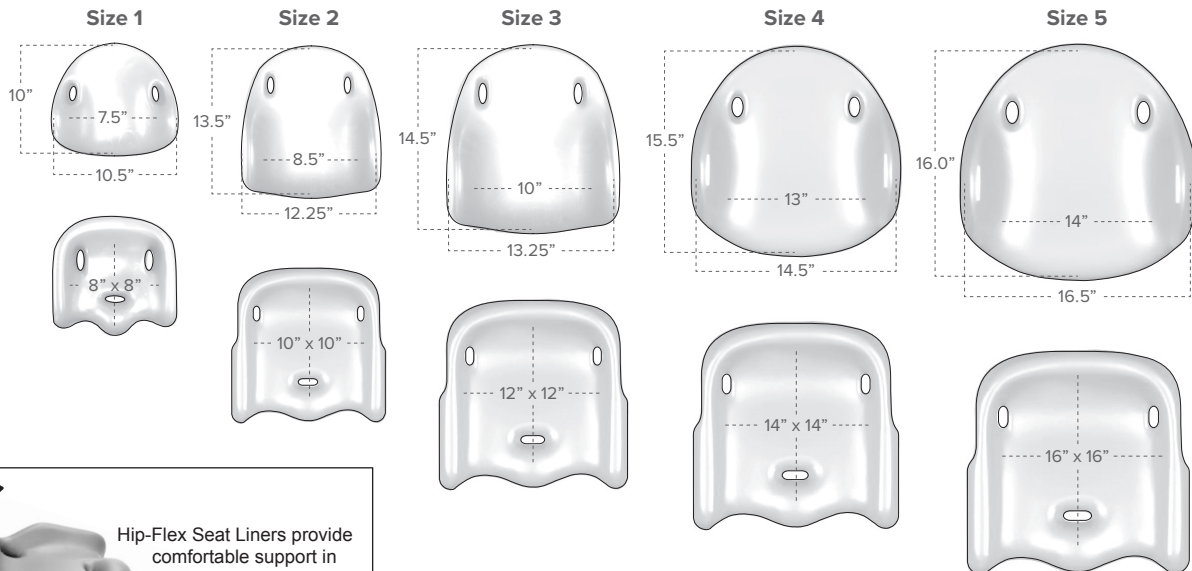

Limited Warranty

Bergeron by Design warrants to the original retail purchaser of the Special Tomato® product, that if any part or component proves defective in material or workmanship within five years of the purchase date, the defective part will be repaired or replaced (at Bergeron Health Care's discretion) free of charge. Warranty service may be performed by an authorized service center or (at Bergeron Health Care's discretion) the factory.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from the dealer the Special Tomato® product was purchased from. The product must be delivered charges pre-paid (UPS recommended) to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem.

Use only Special Tomato® accessories and parts on Special Tomato® products. Special Tomato® parts are not interchangeable with other manufactures' products. Replace any worn parts immediately for safety.

Product Care

Special Tomato Soft-Touch® Products are impermeable to fluids. Use a mild detergent to clean periodically and dry when wet. To extend the life of the hook and loop strips, comb out the hook portion (hard side) to remove lint and other debris.

Customer Service

Contact your dealer directly or contact customer service support:

US & International Phone: 1-315-429-8407

Fax: 1-315-429-8862

⚠️ WARNING: Prevent serious injury or death from falls or sliding out

- Always secure child in the restraint
- Never leave child unattended
- Always keep child in view
- Secure straps tightly
- Keep product away from extreme sources of heat



Bergeron Health Care
15 South Second St
Dolgeville, NY 13329

©2022 Bergeron Health Care



Global Carehab
HOLMSTRUPGAARDVEJ 11
8220 BRABRAND
DENMARK
+45 24 65 55 50



No part of this document may be photocopied, reproduced, or translated to another language without the prior written consent of Bergeron Health Care.