


HEIGHT RIGHTTM chair

Revision 0313
UMEPP03-13LIT

AGES 3 YEARS AND UP

5 EN PARTS

6 EN CHAIR + Early Pelvic Positioner + TRAY

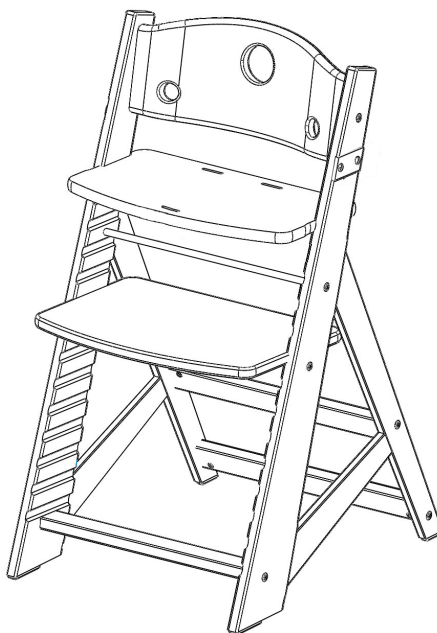

AUTHORISED EUROPEAN REPRESENTATIVE

GLOBAL CAREHAB
ROSENVANGS ALLE 32
8260 VIBY J
DENMARK
+45 24 65 55 50



CONVERSION TO

HEIGHT RIGHTTM chair + Early Pelvic Positioner + Tray



FOR CHILDREN UNDER 3 YEARS OF AGE. ABLE TO SIT INDEPENDENTLY

Height Right™ Chair + Early Pelvic Positioner + Tray

⚠WARNING: For children under 3 years of age able to sit independently. NEVER Use the Tray without the use of the Early Pelvic Positioner (Early Pelvic Positioner and Tray sold separately)

⚠WARNING: Prevent serious injury or death from falls or sliding out.

⚠WARNING: The child should be secured in the chair at all times by the restraining system. The tray is not designed to hold the child in the chair. It is recommended that the chair be used only by children capable of sitting upright unassisted. The Early Pelvic Positioner is ALWAYS required when using the Tray for children under 3 years of age.

⚠WARNING:


- Always secure child in the restraint
- Never leave child unattended
- Always keep child in view
- Secure straps tightly
- Never use as a stepping stool
- The front edge of the footrest should never extend beyond the bottom edge of the front supports at floor level
- Always use chair on smooth level surface

Height Right™ Chair + Early Pelvic Positioner + Tray


Parts

OPTIONAL Early Pelvic Positioner

- 2 long straps (pre installed)
- 1 short strap

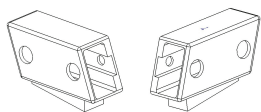


Right + Left Tray Clips

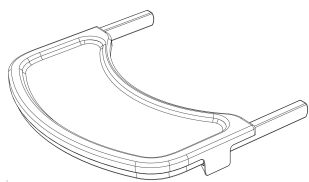

* OPTIONAL TRAY WITH COVER

HEIGHT RIGHT CHAIR

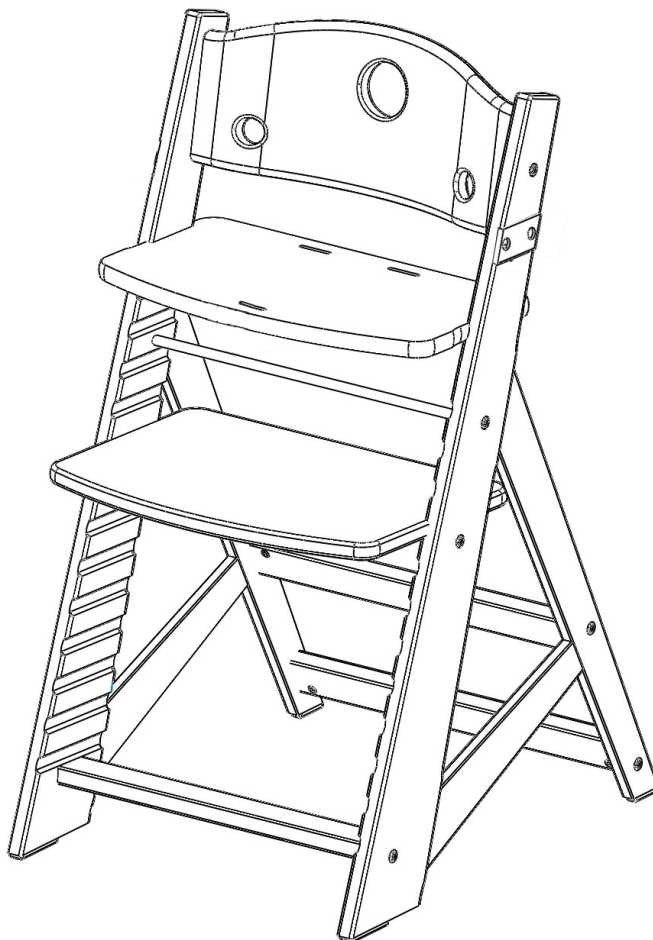

Height Right™ Chair + Early Pelvic Positioner + Tray


CHAIR CONVERSION

Early Pelvic Positioner + Tray Installation

1 Loosen Screws

2 Remove seat plate

3 Using Allen key, Remove top metal cross bar

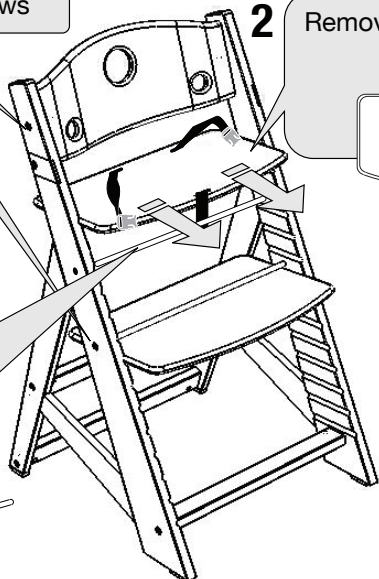


4 Place seat plate flat on table.


Thread restraint straps up through Early Pelvic Positioner

REFERENCE PAGE 3 FOR CHILD RESTRAINT OPERATION

4 Fasten Clips

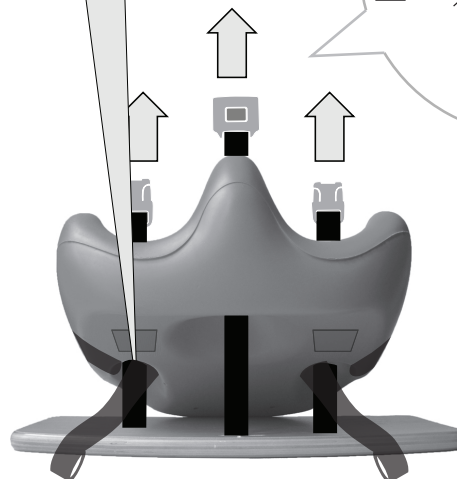


5 Center Early Pelvic Positioner on Seat Plate.

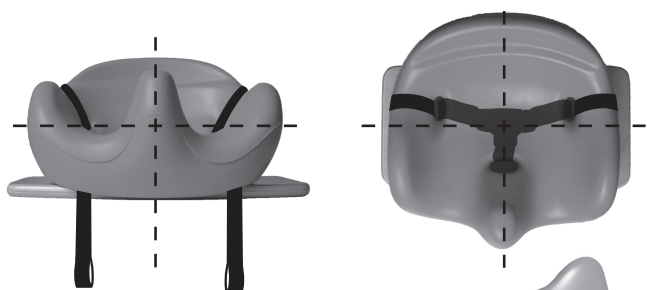

Front lip of Insert will overhang and mate to front edge of seat plate.


6 Keeping the Early Pelvic Positioner centered on the seat plate, lift front edge of insert.

7 Peel off paper to expose adhesive.

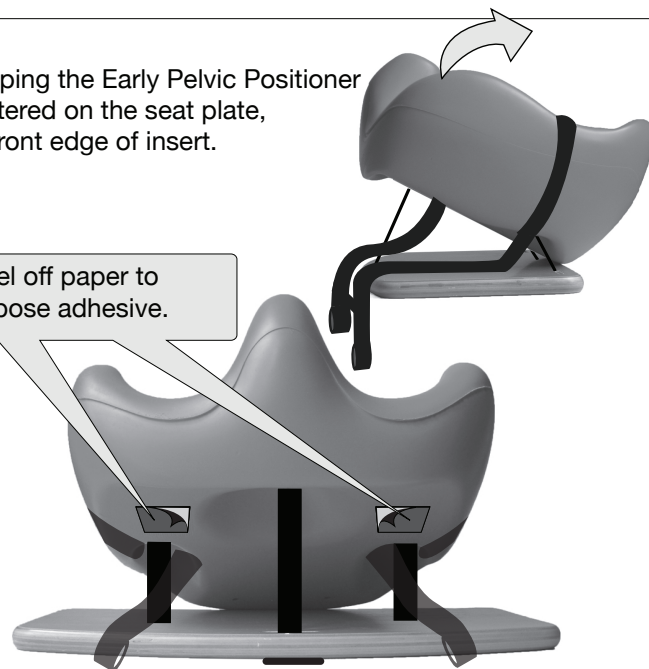


8 Lower front edge back onto seat plate. Adhesive patches will contact wooden seat plate. Do not reposition the cushion for 24 hours to allow adhesive to bond.

Install Small loop strap by sliding long straps through the Smaller loop strap openings.

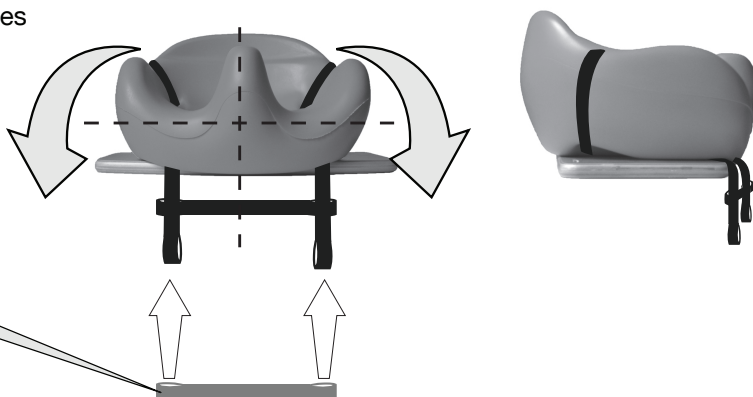

- 9** From the rear of the chair : Slide seat plate and Early Pelvic Positioner in 1ST rung above top metal cross bar until Early Pelvic Positioner mates with backrest.

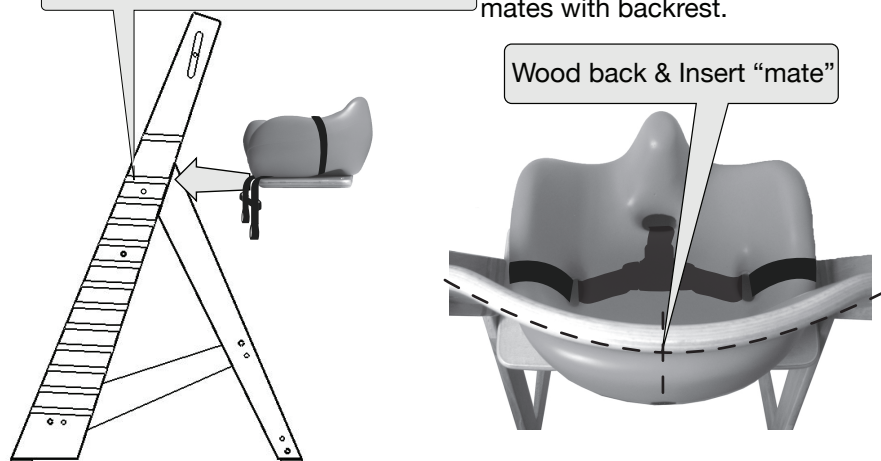

- 10** Thread top metal cross bar through hanging loops under Early Pelvic Positioner seat.




- 11** Reinstall top metal cross bar using original screws.




- 12** Install Tray Clips

Attach the Right Tray Clip as shown with 2 Supplied Screws. Repeat for Left.

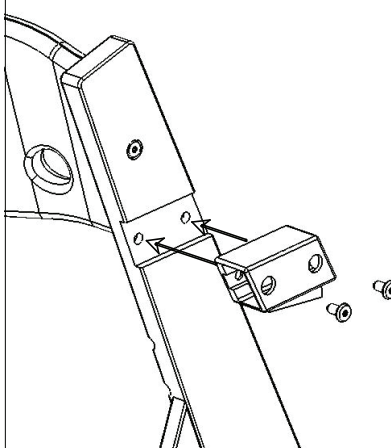


- 13** Installing Tray and using Clips

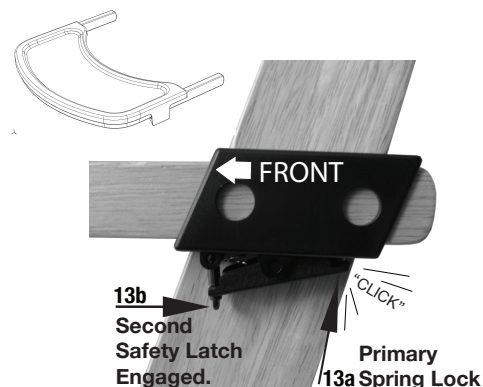

- a** When Tray is properly installed an audible "click" will be heard from primary lock.
b Engage the second safety Latch Lever by pushing it forward into the primary spring lock.

- 14** TIGHTEN ALL SCREWS (BOTH SIDES) IN ORDER 1-5 _DO NOT OVER TIGHTEN!

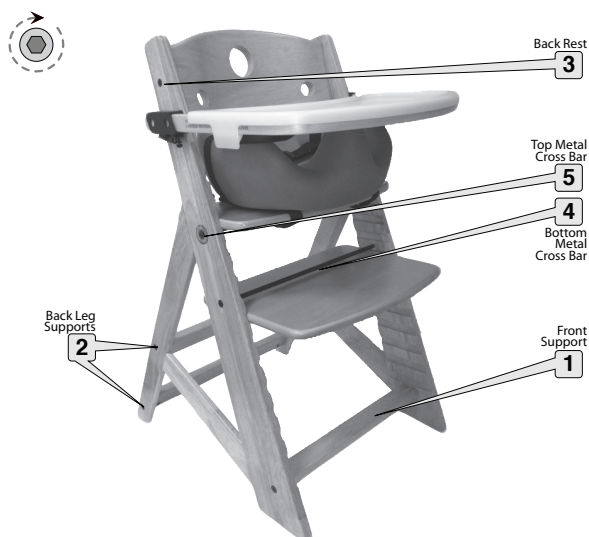

5 Year Limited Warranty

Bergeron by Design warrants to the original retail purchaser of the Height Right Chair product, that if any part or component proves defective in material or workmanship within five years of the purchase date, the defective part will be repaired or replaced (at Bergeron by Design's discretion) free of charge. Warranty service may be performed by an authorized service center or at Bergeron by Design's discretion.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from Bergeron By Design.

The product must be delivered charges pre-paid (UPS recommended) to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem.

Customer Service

Toll Free: 1.866.529.8407
 International: 1.315.429.8407
 Fax: 1.315.429.8862

Product Care

Clean chair with damp cloth. With normal use, over time screws may loosen. Please check regularly. **Only use chair on smooth level surface.**

Notice

Use only Height Right Chair accessories and parts. Height Right Chair parts are not interchangeable with other manufacturers' products. Replace any worn parts immediately for safety. © 2013 Bergeron By Design.